

AEDtrax Instruction Manual

Remove the AED 3 electrode pack from the electrode compartment.

note the serial number of the AED (on the back; starts with "AX....") and the Trax number (on the top; starts with "TH.....")

1



Using a Torx T8 screwdriver, remove the screw of the rubber seal on the back of the AED and dispose of the rubber seal along with the black washer (keep the screw).

2



Plug the supplied USB cable into the back of the AED and reattach the previously removed screw.

3



AEDtrax Instruction Manual

Clean the right edge of the AED housing (viewed from the back) with a degreasing agent (e.g., cleaning benzine).

4



Remove the protective film from the cable holder and attach it to the edge of the AED housing.

5



Open the rubber seal on trax and turn it on using a thin metal pin (e.g., a paperclip)

middle position: trax ON, GPS OFF

left position (recommended): trax & GPS ON

right position: trax off


6



AEDtrax Instruction Manual

Pull the sliding plate on trax upwards, place trax against the electrode compartment, and then push the sliding plate down until it clicks into place and connects with the AED.

7

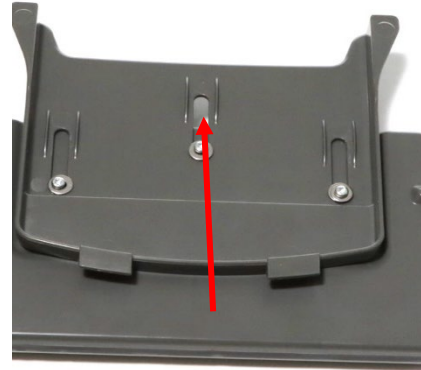

Insert the USB-C plug into trax and check for a clean cable arrangement.

8



Place the electrodes back into the electrode compartment (scissors facing the AED, emergency kit facing away from the AED).

9



AEDtrax Instruction Manual

Configure the AED:

Power ZOLL AED 3 device on to non-rescue mode. Press and hold "ON/OFF" button for 10 seconds or until you see the ZOLL Logo.

10



Press the "Device Configuration" icon to open the configuration screen.

Press the "Supervisor Access" icon.

11



Enter the 6-digit code from the User manual and confirm with "OK".

Initial Code: 5, 6, 7, 8, 9 0

12



AEDtrax Instruction Manual


Press the "Settings" icon.

13




Press "Advanced".
Set "USB Self-Test Push Active" to "ON" and confirm with "OK" and "Save".

14

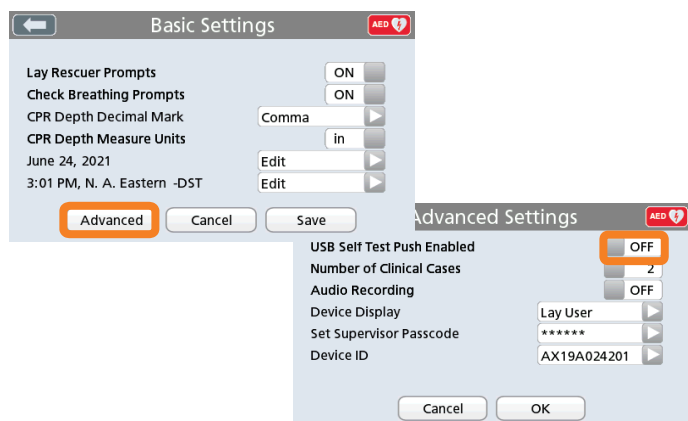

Press the "Export Files" icon.
Select "Device History Data" and press "Save (USB)".
If "Completed" appears at the end, the transfer was successful. Confirm with "OK".

15

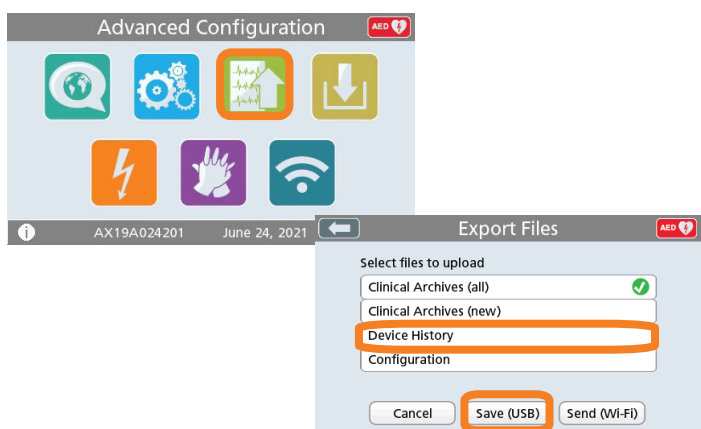

AEDtrax Instruction Manual


Turn off the AED 3 by pressing the power button for approximately 2 seconds. The message "Shutdown in progress" will appear on the monitor.

16



Log in to plustrac to set your device up on the platform.

17

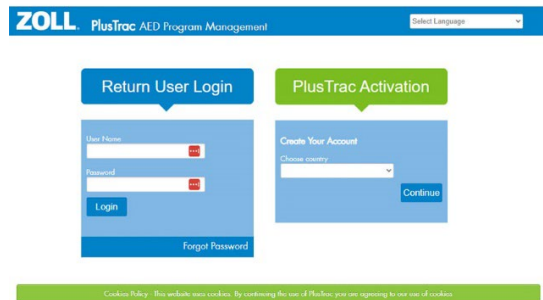

If you are using AEDtrax with the RFID option, place the RFID card on a solid surface. If the surface is metallic, use a spacer of at least 5 mm from the RFID card. Place the RFID card in the cabinet so that it can be connected to the top left corner of the Trax when it is placed in the cabinet.

18

